




# Health and Medical Supply




# MASKS

## N95-MASKS



**Alpha Pro Tech**  
**N-95 Particulate Respirator**  
PFL® (Positive Facial Lock®) Design  
with Magic Arch® and Twist Seal®




**Donning Instructions**



**Step 1.**  
Grasp suspension at top of mask with one hand. Slide top of forehead behind strap piece.



**Step 2.**  
Open mask by pulling cover out and set strap pieces with mask as they open. It is NIOSH APPROVED that mask is fully open.



**Step 3.**  
Place strap in pocket created. Stretch fabric over back of head, positioning behind head behind neck behind ears and top band near crown of head.




**Step 4.**  
Carefully reposition to ensure seal conforms properly to the Magic Arch™. Fully inhale once every three to five seconds.




**Step 5.**  
Holding mask at crown, pinch strap and tug to ensure the mask on the forehead stretches on the skin, pull cover gently so that mask is snug against face.



**Step 6.**  
Pinch any excess material together under chin and fold cover or band.




**Step 7.**  
Face Fit Check: To check fit, place both fingers along the edges of the respirator and inhale. Your respirator should cover nose, sides of the face properly. Look back at the respirator edges, central or behind the strap pieces, repeat Steps 3 and 6.



**Step 8.**  
PFL Mask correctly fit to face.

**Warning:** This respirator is not to be used for protection against chemical, biological, radiological, or nuclear hazards. This respirator is not to be used for protection against airborne particulates that are not covered by this approval. This respirator is not to be used for protection against airborne particulates that are not covered by this approval.

**NIOSH Approval:** This respirator is approved for use against airborne particulates that are not covered by this approval. This respirator is not to be used for protection against chemical, biological, radiological, or nuclear hazards. This respirator is not to be used for protection against airborne particulates that are not covered by this approval.



Customer Service - Phone: 800.749.1363 • Fax: 905.478.8703 • sales@alphaprotech.com



## ALPHA PROTECH CUSTOM FIT NIOSH MASK MODEL 695

**NIOSH Approved**, our N-95 delivers the appropriate amount of filtration efficiency and when combined with a trio of patented and unique design features, also delivers the user a comfortable, custom fitting medical facemask. The Positive Facial Lock® (PFL®) N-95 Particulate Respirator meets the World Health Organization (WHO) and the Centers for Disease Control (CDC) recommended protection levels (where applicable) for many airborne

contaminants.

Terms are prepaid unless otherwise indicated or determined on a case by case basis. Financing is available for most products.

[Inquire for distributor and government pricing](#)

**SKU:** HMS-APT-312

**Price:** \$2.95

**Categories:** [Masks](#), [N95-Masks](#), [Our Products](#)





# **Health and Medical Supply**

