



Health and Medical Supply




OUR PRODUCTS


HealthandMedicalSupply.com LLC is a Florida-based small business dedicated to helping get medical products to those who need them with the highest professional and ethical standards.

MASKS


N95-MASKS




Alpha Pro Tech
N-95 Particulate Respirator
PFL® (Positive Facial Lock®) Design
with Magic Arch® and Twist Seal®




Donning Instructions




Step 1. Grasp top of mask with one hand. Slide top edge of mask over forehead and over nose.




Step 2. Open mask by pulling lower edge and use other hand to pull mask to fully open. If in NIOSH (N95) that mask is fully open.




Step 3. Place over face in pocket position. Bend back over back of head, position, behind head behind neck behind ears and top band over crown of head.




Step 4. Cover nose and mouth to ensure full contact, making it the Magic Arch® to fully extend along your nose and mouth.




Step 5. Adjust mask to ensure proper fit over nose and mouth. Check for any visible obstructions on the mask, and ensure properly fit that mask in NIOSH approved form.



Step 6. Check any necessary sealant together under chin and ear over head.




Step 7. Final Check: To check fit, place both hands along the edges of the respirator and inhale. If air escapes around your nose, edge of the mask piece, or at ends of the respirator straps, adjust or refit the chin piece, repeat steps 5 and 6.



Step 8. PFL. Look closely at face.

Warnings:
 1. This respirator should be used only for the purposes for which it is designed. Changes to the design may affect its performance. Do not use for purposes not intended.
 2. This respirator is not intended for use in environments with oxygen deficiency or in atmospheres containing concentrations of gases or vapors that require the use of a supplied air respirator.
 3. This respirator is not intended for use in atmospheres containing concentrations of gases or vapors that require the use of a supplied air respirator.
 4. This respirator is not intended for use in atmospheres containing concentrations of gases or vapors that require the use of a supplied air respirator.

Notes:
 1. The respirator should be stored in a clean, dry, and well-ventilated area.
 2. The respirator should be stored in a clean, dry, and well-ventilated area.
 3. The respirator should be stored in a clean, dry, and well-ventilated area.
 4. The respirator should be stored in a clean, dry, and well-ventilated area.



Customer Service - Phone: 800.749.1363 • Fax: 905.479.8783 • sales@alphaprotech.com



ALPHA PROTECH CUSTOM FIT NIOSH MASK MODEL 695

NIOSH Approved, our N-95 delivers the appropriate amount of filtration efficiency and when combined with a trio of patented and unique design features, also delivers the user a comfortable, custom fitting medical facemask. The Positive Facial Lock® (PFL®) N-95 Particulate Respirator meets the World Health Organization (WHO) and the Centers for Disease Control (CDC) recommended protection levels (where applicable) for many airborne contaminants.

Terms are prepaid unless otherwise indicated or determined on a case by case basis. Financing is available for most products.

[Inquire for distributor and government pricing](#)

SKU: HMS-APT-312

Price: \$2.95

Categories: [Masks](#), [N95-Masks](#), [Our Products](#)



Health and Medical Supply

