




Health and Medical Supply




MASKS


N95-MASKS




Alpha Pro Tech
N-95 Particulate Respirator
PFL® (Positive Facial Lock®) Design
with Magic Arch® and Twist Seal®




Donning Instructions




Step 1: Grasp suspension at top of mask with one hand. Slide top of forehead behind strap piece.




Step 2: Open mask by pulling over and out on one side until mask is fully open. It is NIOSH APPROVED that mask is fully open.




Step 3: Place strap in pocket created. Stretch bottom over back of head, positioning bottom behind behind neck behind ears and top band near crown of head.




Step 4: Carefully reposition to ensure seal between mask and face contours, making sure Magic Arch™ is fully extended along forehead and around crown of head.




Step 5: Holding mask at crown, pinch strap over bridge of nose. The mask on the forehead should be on the bridge of nose on the bridge of nose.



Step 6: Pinch any excess material together under chin and fold over or tuck.



Step 7: Face Fit Check. To check fit, place both fingers along the edges of the respirator and exhale. Your respirator should create a seal with the nose piece. If air leaks at the respirator edges, untuck or release the strap pieces, repeat Steps 3 and 6.



Step 8: PFL Mask correctly fit to face.

WARNING: This respirator is not to be used for protection against chemical hazards. It is not to be used for protection against biological hazards. It is not to be used for protection against ionizing radiation. It is not to be used for protection against laser radiation. It is not to be used for protection against electromagnetic interference. It is not to be used for protection against fire. It is not to be used for protection against electrical hazards. It is not to be used for protection against mechanical hazards. It is not to be used for protection against noise. It is not to be used for protection against vibration. It is not to be used for protection against heat. It is not to be used for protection against cold. It is not to be used for protection against oxygen deficiency. It is not to be used for protection against carbon monoxide. It is not to be used for protection against other atmospheric hazards. It is not to be used for protection against other respiratory hazards. It is not to be used for protection against other environmental hazards. It is not to be used for protection against other hazards. It is not to be used for protection against other hazards. It is not to be used for protection against other hazards.

AlphaProTech
INDUSTRIES

Customer Service - Phone: 800.749.1363 • Fax: 905.478.8703 • sales@alphaprotech.com



ALPHA PROTECH CUSTOM FIT NIOSH MASK MODEL 695

NIOSH Approved, our N-95 delivers the appropriate amount of filtration efficiency and when combined with a trio of patented and unique design features, also delivers the user a comfortable, custom fitting medical facemask. The Positive Facial Lock® (PFL®) N-95 Particulate Respirator meets the World Health Organization (WHO) and the Centers for Disease Control (CDC) recommended protection levels (where applicable) for many airborne

contaminants.

Terms are prepaid unless otherwise indicated or determined on a case by case basis. Financing is available for most products.

[Inquire for distributor and government pricing](#)

SKU: HMS-APT-312

Price: \$2.95

Categories: [Masks](#), [N95-Masks](#), [Our Products](#)



Health and Medical Supply

