



Health and Medical Supply




OUR PRODUCTS


HealthandMedicalSupply.com LLC is a Florida-based small business dedicated to helping get medical products to those who need them with the highest professional and ethical standards.

MASKS


N95-MASKS




Alpha Pro Tech
N-95 Particulate Respirator
PFL® (Positive Facial Lock®) Design
with Magic Arch® and Twist Seal®




Donning Instructions




Step 1: Check suspension at top of mask with one hand. Grab by finger of other hand behind the pleats.




Step 2: Open mask by pulling lower edge and use other piece until mask is fully open. If in NIOSH (N95) that mask is fully open.




Step 3: Place over in pocket position. Bunch back over back of head, position, behind head behind neck behind ears and top band over crown of head.




Step 4: Cover nosepiece to ensure face conforms, making it the Magic Arch®. In full extension, ensure nose piece and mouth.




Step 5: Push mask all nose piece, place over bridge of nose. Press on the underside between the piece, and ensure gently to that mask is fully sealed.



Step 6: Check any excess material together under chin and cut over at head.



Step 7: Face Fit Check. To check fit, place both hands along the edges of the respirator and exhale. If air escapes around your nose, top of the nose piece, or at ends of the respirator straps, adjust or reseat the chin piece, repeat steps 5 and 6.



Step 8: PFL. Bunch correctly fit to face.

Warnings:
 1. This respirator should be used only for the purposes for which it is designed. Change filters and/or straps when they become damaged or worn.
 2. Do not use this respirator if you have a cold, sore throat, or other respiratory infection.
 3. This respirator is not intended for use in environments with oxygen deficiency.
 4. This respirator is not intended for use in environments with air contaminants that are not listed on the manufacturer's label. Check labels for other restrictions.
 5. Do not use this respirator if you have a facial hair that interferes with the seal.

Notes:
 1. This respirator is not intended for use in environments with oxygen deficiency.
 2. Do not use this respirator if you have a cold, sore throat, or other respiratory infection.
 3. This respirator is not intended for use in environments with air contaminants that are not listed on the manufacturer's label. Check labels for other restrictions.
 4. Do not use this respirator if you have a facial hair that interferes with the seal.

AlphaProTech
 LABORATORIES

Customer Service - Phone: 800.749.1363 • Fax: 905.479.8783 • sales@alphaprotech.com



ALPHA PROTECH CUSTOM FIT NIOSH MASK MODEL 695

NIOSH Approved, our N-95 delivers the appropriate amount of filtration efficiency and when combined with a trio of patented and unique design features, also delivers the user a comfortable, custom fitting medical facemask. The Positive Facial Lock® (PFL®) N-95 Particulate Respirator meets the World Health Organization (WHO) and the Centers for Disease Control (CDC) recommended protection levels (where applicable) for many airborne contaminants.

Terms are prepaid unless otherwise indicated or determined on a case by case basis. Financing is available for most products.

[Inquire for distributor and government pricing](#)

SKU: HMS-APT-312

Price: \$2.95

Categories: [Masks](#), [N95-Masks](#), [Our Products](#)



Health and Medical Supply

