




# Health and Medical Supply




# MASKS


## N95-MASKS




**Alpha Pro Tech**  
**N-95 Particulate Respirator**  
PFL® (Positive Facial Lock®) Design  
with Magic Arch® and Twist Seal®




**Donning Instructions**




**Step 1.**  
Grasp suspension at top of mask with one hand. Other hand at forehead behind strap piece.




**Step 2.**  
Open mask by pulling over one ear and on the other side with mask is fully open. It is NIOSH APPROVED that mask is fully open.




**Step 3.**  
Place strap in pocket created. Stretch bottom over back of head, positioning bottom behind behind neck behind ears and top band near crown of head.




**Step 4.**  
Carefully reposition to ensure seal conforms properly to the Magic Arch™. Fully inhale every time to ensure seal.




**Step 5.**  
Holding mask at crown, pinch strap over bridge of nose. The mask on the forehead should be on the strap, pull strap gently so that mask is snug against face.



**Step 6.**  
Pinch any excess material together under chin and fold over or tuck.




**Step 7.**  
Face Fit Check: To check fit, pinch both straps along the edges of the respirator and exhale. Your respirator should cover most of the nose, mouth and chin. If the respirator leaks, adjust or replace the strap pieces. Repeat Steps 3 and 6.



**Step 8.**  
PFL Mask correctly fit to face.

**WARNING:** This respirator is not to be used for protection against chemical hazards. It is not to be used for protection against biological hazards. It is not to be used for protection against ionizing radiation. It is not to be used for protection against laser radiation. It is not to be used for protection against electromagnetic interference. It is not to be used for protection against fire. It is not to be used for protection against explosion. It is not to be used for protection against electrical shock. It is not to be used for protection against electrical fire. It is not to be used for protection against electrical explosion. It is not to be used for protection against electrical fire. It is not to be used for protection against electrical explosion.

**NIOSH:** This respirator is not to be used for protection against chemical hazards. It is not to be used for protection against biological hazards. It is not to be used for protection against ionizing radiation. It is not to be used for protection against laser radiation. It is not to be used for protection against electromagnetic interference. It is not to be used for protection against fire. It is not to be used for protection against explosion. It is not to be used for protection against electrical shock. It is not to be used for protection against electrical fire. It is not to be used for protection against electrical explosion.



Customer Service - Phone: 800.749.1363 • Fax: 905.478.8703 • sales@alphaprotech.com



## ALPHA PROTECH CUSTOM FIT NIOSH MASK MODEL 695

**NIOSH Approved**, our N-95 delivers the appropriate amount of filtration efficiency and when combined with a trio of patented and unique design features, also delivers the user a comfortable, custom fitting medical facemask. The Positive Facial Lock® (PFL®) N-95 Particulate Respirator meets the World Health Organization (WHO) and the Centers for Disease Control (CDC) recommended protection levels (where applicable) for many airborne

contaminants.

Terms are prepaid unless otherwise indicated or determined on a case by case basis. Financing is available for most products.

[Inquire for distributor and government pricing](#)

**SKU:** HMS-APT-312

**Price:** \$2.95

**Categories:** [Masks](#), [N95-Masks](#), [Our Products](#)





# **Health and Medical Supply**

