





Health and Medical Supply



N95-MASKS




Alpha Pro Tech
N-95 Particulate Respirator
 PFL® (Positive Facial Lock®) Design
 with Magic Arch® and Twist Seal®




Donning Instructions


Step 1. Grasp suspension at top of mask with one hand. Allow the top of mask to rest on the other hand.




Step 2. Open mask by pulling lower edge and use other hand to hold mask as fully open. If a MAGIC ARCH® that mask is fully open.




Step 3. Place one ear in pocket holder. Stretch across over back of head, positioning holder back behind neck holder over and top head cover corner of head.




Step 4. Continue response to suspension from ear hooks, making sure Magic Arch® is fully extended along your nose and mouth.




Step 5. Tilt the mask so that the Magic Arch® is fully extended across the bridge of the nose and the Twist Seal® is fully extended across the chin.




Step 6. If any excess material is present, fold it over and tuck it into the mask.



Step 7. Final Check: To check fit, place both hands along the edges of the mask and inhale. If air escapes around your nose, side of the mask or chin, repeat steps 1-6.



Step 8. PFL Mask correctly fit to face.



Warnings:

1. This respirator does not protect the user from hazardous air pollutants or other contaminants. These respirators are not intended to be used for protection against chemical, biological, radiological, or nuclear hazards.

2. Failure to follow the instructions for use of this respirator may result in reduced protection.

3. This respirator is not intended for use in environments where the concentration of contaminants exceeds the NIOSH approved level.

4. This respirator is not intended for use in environments where the concentration of contaminants exceeds the NIOSH approved level.

5. This respirator is not intended for use in environments where the concentration of contaminants exceeds the NIOSH approved level.

6. This respirator is not intended for use in environments where the concentration of contaminants exceeds the NIOSH approved level.

7. This respirator is not intended for use in environments where the concentration of contaminants exceeds the NIOSH approved level.

8. This respirator is not intended for use in environments where the concentration of contaminants exceeds the NIOSH approved level.

9. This respirator is not intended for use in environments where the concentration of contaminants exceeds the NIOSH approved level.

10. This respirator is not intended for use in environments where the concentration of contaminants exceeds the NIOSH approved level.

AlphaProTech
 L.L.P.

Customer Service - Phone: 800.749.1363 • Fax: 905.479.8783 • sales@alphaprotech.com



ALPHA PROTECH CUSTOM FIT NIOSH MASK MODEL 695

NIOSH Approved, our N-95 delivers the appropriate amount of filtration efficiency and when combined with a trio of patented and unique design features, also delivers the user a comfortable, custom fitting medical facemask. The Positive Facial Lock® (PFL®) N-95 Particulate Respirator meets the World Health Organization (WHO) and the Centers for Disease Control (CDC) recommended protection levels (where applicable) for many airborne contaminants.

Terms are prepaid unless otherwise indicated or determined on a case by case basis. Financing is available for most products.

[Inquire for distributor and government pricing](#)

SKU: HMS-APT-312

Price: \$2.95

Categories: [Masks](#), [N95-Masks](#), [Our Products](#)



Health and Medical Supply

