



Health and Medical Supply




OUR PRODUCTS


HealthandMedicalSupply.com LLC is a Florida-based small business dedicated to helping get medical products to those who need them with the highest professional and ethical standards.

MASKS


N95-MASKS




Alpha Pro Tech
N-95 Particulate Respirator
PFL® (Positive Facial Lock®) Design
with Magic Arch® and Twist Seal®




Donning Instructions




Step 1: Check suspension at top of mask with one hand. Grab by finger of other hand behind the pleats.




Step 2: Open mask by pulling lower edge and use other piece until mask is fully open. If in NIOSH (N95) that mask is fully open.




Step 3: Place over in pocket position. Bunch back over back of head, position, behind head behind neck behind ears and top band over crown of head.




Step 4: Cover nosepiece to ensure face contact, making it the Magic Arch™ to fully ensure snug face mask seal.




Step 5: Push mask all nose piece, place over bridge of nose. Press on the underside between the piece, and lower gently to that mask is fully seated.



Step 6: Check any excess material together under chin and cut over head.




Step 7: Face Fit Check. To check fit, place both hands along the edges of the respirator and exhale. If air escapes around your nose, top of the nose piece, or at ends of the respirator straps, adjust or refit the chin piece, repeat steps 3 and 4.



Step 8: PFL. Make sure PFL is to face.

Warnings:
 1. This product does not protect the user from hazardous materials or biological agents. Change filters and/or headbands when they become wet or soiled. Wipe the face with 70% alcohol before use.
 2. Do not use if you have a fever, sore throat, or other respiratory symptoms.
 3. Do not use if you have a fever, sore throat, or other respiratory symptoms.
 4. Do not use if you have a fever, sore throat, or other respiratory symptoms.

Notes:
 1. This product is not to be used for protection against chemical, biological, radiological, or nuclear hazards.
 2. This product is not to be used for protection against chemical, biological, radiological, or nuclear hazards.
 3. This product is not to be used for protection against chemical, biological, radiological, or nuclear hazards.
 4. This product is not to be used for protection against chemical, biological, radiological, or nuclear hazards.



Customer Service - Phone: 800.749.1363 • Fax: 905.479.8783 • sales@alphaprotech.com



ALPHA PROTECH CUSTOM FIT NIOSH MASK MODEL 695

NIOSH Approved, our N-95 delivers the appropriate amount of filtration efficiency and when combined with a trio of patented and unique design features, also delivers the user a comfortable, custom fitting medical facemask. The Positive Facial Lock® (PFL®) N-95 Particulate Respirator meets the World Health Organization (WHO) and the Centers for Disease Control (CDC) recommended protection levels (where applicable) for many airborne contaminants.

Terms are prepaid unless otherwise indicated or determined on a case by case basis. Financing is available for most products.

[Inquire for distributor and government pricing](#)

SKU: HMS-APT-312

Price: \$2.95

Categories: [Masks](#), [N95-Masks](#), [Our Products](#)



Health and Medical Supply

